DE	Achillessehnen-Massagegerät Gebrauchsanweisung	IT	Apparecchio per massaggi per il tendine d'Achille Istruzioni per l'uso
EN	Achilles tendon massager Instruction for Use	TR	Aşil tendonu masaj cihazı Kullanım kılavuzu
FR	Appareil de massage du tendon d'Achille Mode d'emploi.....	RU	Прибор для массажа ахиллова сухожилия Инструкция по применению.....
ES	Aparato de masaje del tendón de Aquiles Instrucciones de uso	PL	Urządzenie do masażu ścięgna Achillesa Instrukcja obsługi.....

Inhalt

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Contents

1. Intended use	12	6. Cleaning and storage	18
2. Safety notes	13	7. What if there are problems?	18
3. Device description	15	8. Technical data	19
4. Initial use	15	9. Warranty/service	19
5. Use	16		

Read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

Included in delivery


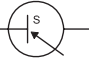
- Achillomed®
- Mains part
- These instructions for use

WARNING

- The device is intended only for domestic/private use, not for commercial use.
- This device can be used by children over the age of eight and by people with reduced physical, sensory or mental skills or a lack of experience or knowledge, provided that they are supervised or have been instructed on how to use the device safely and are aware of the consequent risks of use.
- Children must not play with the device.
- Cleaning and user maintenance must not be performed by children unless supervised.
- If the mains connection cable of this device is damaged, it must be disposed of. If it cannot be removed, the device must be disposed of.

Signs and symbols

The following symbols appear in the instructions for use and on the type plate:

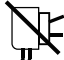	Warning Warning instruction indicating a risk of injury or damage to health		Manufacturer
	Important Safety note indicating possible damage to the device/accessory		CE labelling This product satisfies the requirements of the applicable European and national directives.
	Product information Note on important information		United Kingdom Conformity Assessed Mark
	Only use indoors.		The products demonstrably meet the requirements of the Technical Regulations of the EAEU.
	Observe the instructions for use		Safety isolating transformer, short-circuit proof
	Green dot (Der Grüne Punkt): German dual waste collection system		Energy efficiency level 6
	Dispose of packaging in an environmentally friendly manner		Power supply unit, switch mode; switch mode power supply unit; SMPS
	Protection class 2 device	IP 30	Device protected against foreign objects ≥ 2.5 mm
	Disposal in accordance with the Waste Electrical and Electronic Equipment EC Directive – WEEE		Do not use the power adapter if the plug is damaged.
	Polarity of d.c. power connector		

1. Intended use

This device is exclusively intended to massage the human Achilles tendon. The device is only suitable for **private use**, and not for medical or commercial purposes. It cannot replace medical treatment. Do not use the device if one or several of the following warnings applies to you. If you are unsure whether the device is suitable for you, please consult your doctor. The device is only intended for the purpose described in these instructions for use. The manufacturer is not liable for damage resulting from improper or careless use.

2. Safety notes

Read these instructions for use carefully! Non-observance of the following information may result in personal injury or material damage. Store these instructions for use and make them accessible to other users. Make sure you include these instructions for use when handing over the device to third parties.

WARNING

Do not use the device

- On animals
- If you suffer from a medical abnormality or injury of the Achilles tendon (e.g. open wounds)
- If you have a pacemaker
- During pregnancy
- On the face (eyes), on the larynx or any other particularly sensitive body parts
- For foot massages
- While sleeping
- After the consumption of medicines or alcohol (reduced cognition!)
- In vehicles

You should consult your doctor before using the device, especially

- If you suffer from a severe illness or have recently had surgery on your Achilles tendon
- If you have a pacemaker, implants or other aids
- If you suffer from thrombosis, diabetes or unexplained pain

WARNING

Keep packaging material away from children. There is the risk of suffocation.

Electric shock

WARNING

As with every electrical device, use the Achillomed® carefully and cautiously in order to avoid hazards due to electric shock.

Therefore, observe the following instructions for use:

- Only use the device with the mains part provided and at the mains voltage specified on the mains part
- Never use the device if it or its accessories show visible signs of damage
- Do not use the device during a thunderstorm

In the event of defects or operational faults, switch the device off immediately and disconnect it from the mains socket. Do not pull on the mains cable or on the device to disconnect the mains part from the socket. Never hold or carry the device by the mains cable. Keep the mains cable away from hot surfaces. Do not crush, bend or twist the mains cable. Do not insert needles or other sharp objects into the device.

Ensure that the device, the mains part and the mains cable do not come into contact with water, steam or other liquids. Only use the device in dry rooms (e.g. never in the bath, sauna).

Never reach for a device that has been submerged in water. Unplug the mains part from the socket immediately. Do not use the device if it or its accessories are visibly damaged.

Do not drop the device or subject it to any impacts.

Repairs

WARNING

- Only specialist personnel may perform repairs on electrical devices. Improper repairs may subject users to considerable danger. For repairs, please contact Customer Services or an authorised retailer.

Risk of fire

WARNING

There is a risk of fire if the device is used improperly or these instructions for use are ignored.

Therefore, observe the following instructions for use:

- Never use the massager underneath a cover, such as a blanket, pillow, etc.
- Never use the device near petrol or other highly flammable substances.

Handling

IMPORTANT

The device must be switched off and disconnected from the mains after every use and before cleaning.

- Do not place any objects between the rotating massage heads. Ensure that the moving parts can move freely at all times.
- Do not sit, lie or stand with your full weight on the moving parts of the device.
- Do not place any objects on top of the device.
- Do not expose the device to high temperatures.

Disposal

IMPORTANT


For environmental reasons, do not dispose of the device in the household waste at the end of its service life.

Dispose of the device at a suitable local collection or recycling point. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment).

If you have any questions, please contact the local authorities responsible for waste disposal.


3. Device description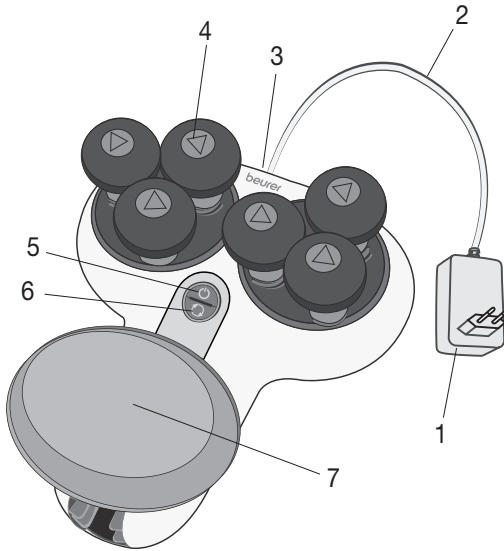

1. Mains part
2. Mains cable
3. Mains part socket
4. 2x3 rotating massage heads
5. ⏻ button
Press once = low massage setting
Press twice = high massage setting
Press 3 times = switch off device
6. ↻ button
(2 massage directions)
7. Footrest with height and width adjustment

4. Initial use

- Remove the packaging.
- Check the device, mains part and mains cable for damage.
- Insert the mains cable connector plug into the mains part socket on the device.
- Ensure that the mains cable does not pose a trip hazard.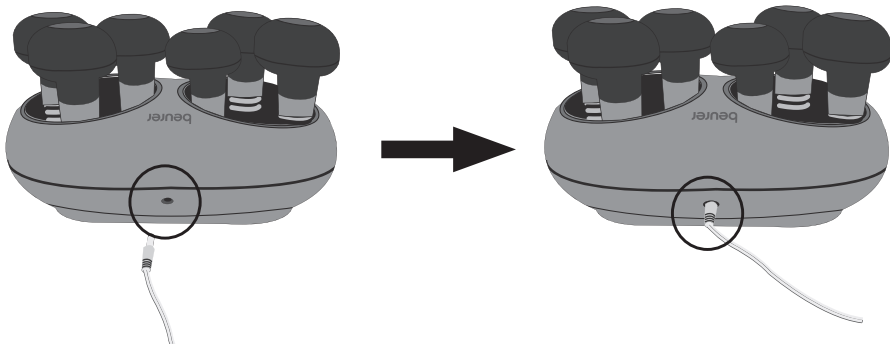

5. Use


Note


- Do not use the Achillomed® for longer than 15 minutes a day. Longer massages can lead to over-stimulation of the Achilles tendon, creating tension instead of aiding relaxation.
- The Achillomed® features an automatic switch-off function, which is set to the maximum operating duration of 15 minutes. If you change the massage speed or direction during the application, the automatic switch-off timer starts again from the beginning (15 minutes).
- You can use the Achillomed® with bare feet or with socks on. To ensure an optimal massage effect, we recommend that you use the Achillomed® with bare feet.

To start the application with the Achillomed®, proceed as follows:

1. Plug the mains part into a suitable socket. Ensure that the mains cable does not pose a trip hazard.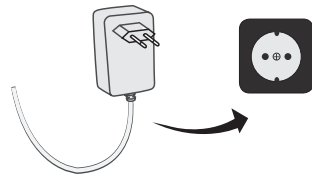


2. Place the Achillomed® on a raised surface. Ensure that the Achillomed® is positioned roughly at the height of your seat (e.g. chair or sofa).

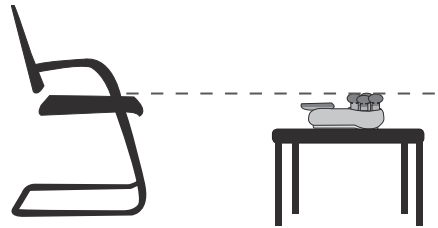3. Sit down on the chair (or any kind of seat) in front of the positioned Achillomed®.


WARNING Trip hazard! Never use the Achillomed® when standing up.

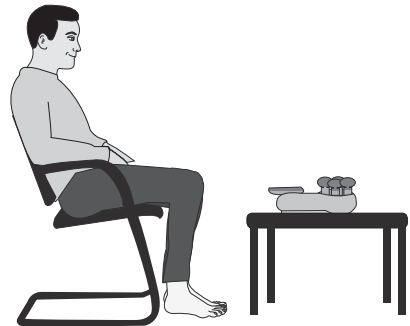4. Press the  button on the Achillomed®. The massage heads start to rotate.


WARNING Danger of crushing! Do not force or jam your hands, fingers or hair between the rotating massage heads.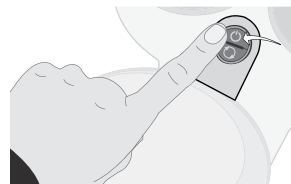

5. Position your leg carefully with the Achilles tendon to be treated on the Achillomed®. Ensure that your Achilles tendon is in the centre between the rotating massage heads. The massage should feel pleasant and relaxing at all times. If the massage is painful or uncomfortable, stop the massage or change the position of your foot.


You can individually adjust the position of your foot using the footrest with height and width adjustment.

You can place your other leg on top of the leg being treated. This way, you can also regulate the massage intensity.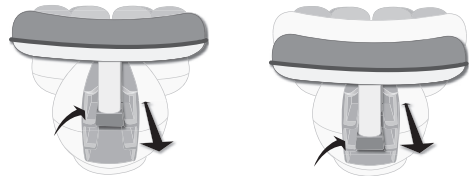


6. To change the massage direction, press the ↻ button. Depending on the massage direction, the left or right LED of the ↻ button lights up.

If you press and hold the ↻ button for half a second, the Achillomed® switches to a massage programme. The left and right LEDs of the ↻ button light up. The massage direction is now controlled automatically. Press the ↻ button to switch the automatic massage programme off again.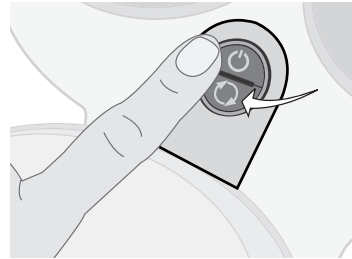

7. To increase the massage speed, press the ⏩ button again. Pressing the ⏩ button again switches the Achillomed® off.

i **Note** We recommend that you start with the lower massage speed so that you get used to the Achillomed® gradually.

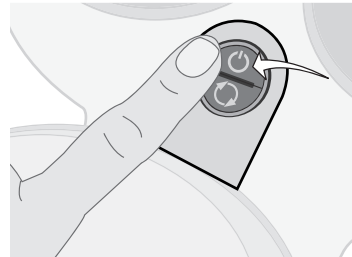
6. Cleaning and storage

Cleaning

WARNING

- Always unplug the mains part from the socket and disconnect it from the device before carrying out any cleaning.
- Clean the device using only the methods specified. Under no circumstances may liquid enter the device or the accessories.
- Small marks can be removed using a cloth or damp sponge, and some liquid delicate laundry detergent where necessary. Do not use any solvent-based cleaning products.
- For hygienic cleaning, you can also remove the massage heads individually. To do this, press the triangle on the massage head and pull it up and off. Clean the massage heads with a damp cloth and some liquid delicate laundry detergent where necessary. Do not use any solvent-based cleaning products. Do not put the cleaned massage heads back on the Achillomed® until they are completely dry.

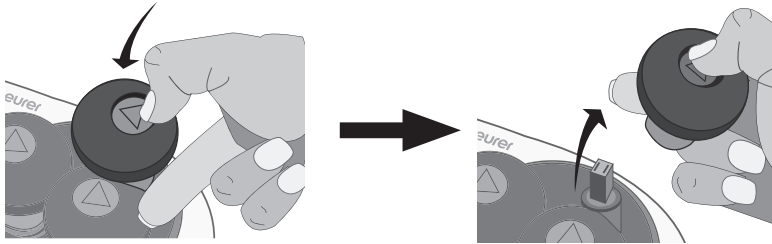- Do not use the device again until it is completely dry.

Storage

If you do not plan to use the device for a long period of time, we recommend that you store it in the original packing in a dry environment, ensuring it is not weighed down by additional items. Do not place any objects on top of the massage heads.

7. What if there are problems?

Problem	Cause	Solution
Massage heads rotating too slowly.	The load on the massage heads is too great.	Free up massage heads.
Massage heads do not move.	The device is not connected to mains power supply.	Plug in the mains part and switch on the device (⏻ button).

8. Technical data

Dimensions	26.5 x 33.5 x 13.4 cm
Weight	approx. 2500 g
Mains operation: Input Output	100-240 V~, 50/60 Hz, 0.5 A 15.0 V === 1,0 A; 15.0 W
Average efficiency during operation	≥ 85.36 %
Efficiency at low load (10%)	≥ 77.44 %
No-load power consumption	≤ 0.045 W
Adapter protection type	IP30
Adapter protection class	II

9. Warranty/service

Further information on the warranty and warranty conditions can be found in the warranty leaflet supplied.

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