beurer zubehörset accessory set

D	Zubehörset.....	2	I	Kit di accessori.....	28
GB	Accessory set.....	8	TR	Aksesuar seti.....	34
F	Kit d'accessoires.....	14	RUS	Набор принадлежностей..	40
E	Set de accesorios.....	21	PL	Zestaw akcesoriów.....	47

Inhalt

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Table of contents

1. Getting to know your device	8
2. Initial use	12
3. Information regarding the positioning of electrodes ..	12
4. Replacement parts and wearing parts	13

1. Getting to know your device

Dear customer,

Thank you for choosing one of our products. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage and air. Please read these brief instructions carefully and keep them for later use. Be sure to make them accessible to other users and observe the information they contain.

With kind regards,
Your Beurer team

WARNING

These electrodes are exclusively intended for use with the EM32, EM37 and EM39 abdominal toning belts.

We strongly advise that you to read the comprehensive instructions for use for the abdominal belt (EM32, EM 37 or EM 39) including all warning and safety notes before using the device for the first time! These brief instructions do not replace the comprehensive instructions for use!

Note

Only use the electrodes on skin that is clean and preferably free from hair and grease in order to extend the life of the electrodes. Worn electrodes may cause skin irritations as they can no longer ensure an even distribution of current.

We recommend replacing the electrodes after having used them 20 times. If the electrodes partially detach before they have been used 20 times, replace them at an earlier stage.

Included in delivery

- 2 x adhesive electrodes (45 x 45 mm)
- 1 x connection cable
- 1 x these brief instructions

PRECAUTIONS

- Always pull gently on the electrodes and cable to remove them from the skin to prevent injuries in the unusual case of highly-sensitive skin.
- Do not expose the electrodes and cable to direct sunlight or high temperatures.
- For hygiene reasons, the electrodes may only be used on one person.
- Switch off the device or the respective channel first before removing or relocating the electrodes to prevent unintentional stimulation.
- Do not modify the electrodes (e.g. by cutting them). This increases the current intensity, which is potentially hazardous (max. recommended output value for the electrodes is 9 mA/cm^2 , an effective current intensity beyond 2 mA/cm^2 requires increased awareness).
- Ensure that no metallic objects (e.g. belt buckles or necklaces) come into contact with the electrodes during stimulation. If you are wearing jewellery or have piercings in the area to be treated (e.g. a navel piercing), these must be removed before using the device. Failure to do so could result in spot burns.
- Keep the electrodes and cable away from children.
- Make sure not to confuse the electrode cables including contacts with your headphones or other devices and do not connect the electrodes to other devices.
- Do not use the EMS device while using other devices that transmit electrical impulses into your body.

- Treatment of the skin with moisturising lotions or ointments beforehand is not recommended as this considerably increases the electrode wear and may cause unpleasant current peaks.
- The electrodes and cable are not intended for use by children or people with restricted physical, sensory (e.g. reduced sensitivity to pain) or mental skills or a lack of experience and/or lack of knowledge, unless they are supervised by a person who is responsible for their safety or are instructed by such a person in how to use the device.
- If the electrodes and cable are damaged, do not use them and contact your retailer or the specified Customer Services address.
- Check the electrodes and cable for signs of wear and tear or damage. If there are such signs of wear and tear or damage or if the electrodes and cable have been used improperly, they must be returned to the manufacturer or retailer before further use.

2. Initial use

1. Connect the connection cable with the electrodes (Fig. 1).

i The electrodes feature clip closures to make connection particularly easy.

2. Guide the connection cable plugs into the socket on the side of the device.

3. Do not pull, twist or make sharp bends in the cables.

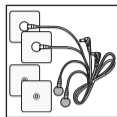
Fig. 1

3. Information regarding the positioning of electrodes

It is fundamental to the intended success of electrostimulation applications that electrodes are positioned sensibly.

Electrode distance:

The greater the distance between the electrodes, the larger the stimulated tissue volume. This applies to the area and depth of the tissue volume. At the same time, however, the stimulation intensity of the tissue falls the further the electrodes are apart. As a result, greater distances between electrodes mean a larger tissue volume is stimulated, but less intensively. Consequently, you must increase the impulse intensity to boost stimulation.

The following guideline applies to selection of the electrode distance:

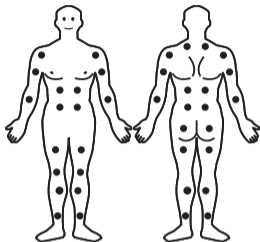
- Sensible distance: approx. 5 – 15 cm
- At distances below 5 cm, the device primarily stimulates surface structures intensively

- At distances in excess of 15 cm, large areas and deep structures are stimulated very weakly

Possible electrode positions:

Position the electrodes so that they surround the affected muscular segments. For optimum effect, do not exceed an electrode distance of approximately 15 cm.

Do not apply the electrodes to the front wall of the chest, i.e. do not massage the large left and right pectoral muscles.

4. Replacement parts and wearing parts

Replacement parts and wearing parts are available from the corresponding listed service address under the stated material number.

Designation	Item number and/or order number
8 x adhesive electrodes (45 x 45)	661.02
4 x adhesive electrodes (50 x 100)	661.01

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Beurer GmbH • Söflinger Straße 218 • 89077 Ulm, Germany