
beurer

HA 55 Pair





DE	Hörhilfen	
	Gebrauchsanweisung	4
EN	Hearing amplifiers	
	Instructions for use.....	25
FR	Aides auditives	
	Mode d'emploi.....	45
ES	Audífonos	
	Instrucciones de uso	65

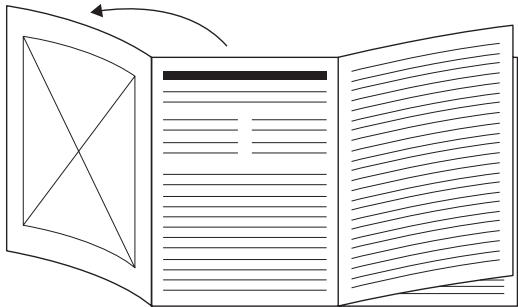CE 1639

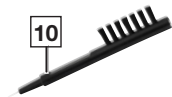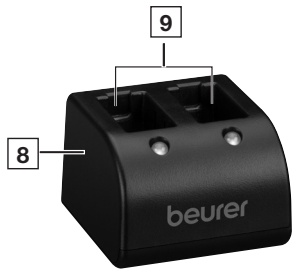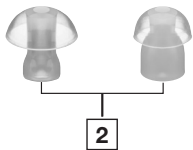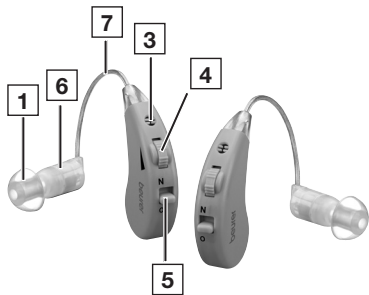- DE** Klappen Sie vor dem Lesen der Gebrauchsanweisung die Seite 3 aus.
- EN** Unfold page 3 before reading the instructions for use.
- FR** Dépliez la page 3 avant de lire le mode d'emploi.
- ES** Despliegue la página 3 antes de leer las instrucciones de uso.

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Read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

Table of contents

1. Included in delivery	26
2. Signs and symbols	26
3. Warnings and safety notes	28
4. Intended use	33
5. Device description	34
6. Initial use	35
7. Cleaning and maintenance	37
8. Accessories and replacement parts	39
9. What if there are problems?	39
10. Disposal	42
11. Technical specifications	43

1. INCLUDED IN DELIVERY

Check that the exterior of the cardboard delivery packaging is intact and make sure that all contents are present. Before use, ensure that there is no visible damage to the devices or accessories and that all packaging material has been removed. If you have any doubts, do not use the device and contact your retailer or the specified Customer Service address.

2 x HA 55 Hearing amplifiers

6 x earplugs (in different sizes and designs)

1 x cleaning brush

1 x drying capsule

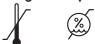
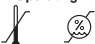
1 x storage box

1 x instructions for use

2. SIGNS AND SYMBOLS

The following symbols are used on the device, in these instructions for use, on the packaging and on the type plate for the device:

 WARNING	Warning notice indicating a risk of injury or damage to health.
 IMPORTANT	Safety note indicating possible damage to the device.
 Note	Note on important information.

	Observe the instructions for use.
	Serial number
	Application part type B
	Disposal in accordance with the Waste Electrical and Electronic Equipment EC Directive – WEEE
	CE labelling This product satisfies the requirements of the applicable European and national directives.
	European Authorized Representative for manufacturers of medical devices
	Manufacturer
<p>Storage/Transport</p> 	Permissible storage and transport temperature and humidity
<p>Operating</p> 	Permissible operating temperature and humidity
	Dispose of packaging in an environmentally friendly manner.

IP21	Protected against foreign objects ≥ 12.5 mm and against vertically falling drops of water
	Use by

3. WARNINGS AND SAFETY NOTES

WARNING

- You must have your hearing checked by a doctor or qualified hearing care professional before using the hearing amplifiers. The settings of the hearing amplifiers should only be adjusted by a trained expert, for example an ENT specialist, an audiologist or a qualified hearing amplifier retailer.
- In the event of an existing or previous ear problem, you should consult your doctor before using the hearing amplifiers.
- Use for patients suffering from an ear inflammation or infection is NOT permissible.
- Do NOT use the hearing amplifiers if you are suffering from an inflammation of the middle ear or ear canal.
- If you feel unwell or experience pain when wearing the hearing amplifiers, stop using them immediately and contact your doctor.
- If your ear becomes inflamed or your skin irritated or you experience an increased accumulation of earwax when wearing the hearing amplifiers, stop using them immediately and contact your doctor.

- Stop using the hearing amplifiers and seek medical attention if the following occurs:
 - Rash or eczema on areas of skin that come into contact with the hearing amplifiers. (The hearing amplifiers are made from a material that is kind to the skin. In rare cases, however, allergic reactions may occur that appear in the form of eczema.)
 - Use of the hearing amplifiers results in discharge from the ear and/or necessitates ear treatment.
 - Sudden, significant decrease in hearing ability.
- If parts of the hearing amplifier become stuck in the ear, seek medical attention immediately. Do not attempt to remove the parts yourself.
- At first, set the volume to a low setting and increase it gradually once you have become accustomed to the hearing amplifiers.
- Do NOT set the volume button too high, as this may cause further damage to your hearing over an extended period of time.
- Do NOT lend your hearing amplifiers to others or use the hearing amplifiers of others, as there is a risk of hearing damage and infection.
- Do NOT pass on your hearing amplifiers for use by other persons. The use of a hearing amplifier that was designed for someone else may cause hearing damage.
- The hearing amplifiers are NOT suitable for use by small children below the age of 36 months.

- It is particularly difficult to diagnose hearing problems in children. Children should only be provided with a hearing amplifier following a thorough examination by an ENT specialist.
- Keep the hearing amplifiers, the accessories and the batteries away from children, as there is a risk of injury and choking.
- This device is NOT intended for use by people (including children) with restricted physical, sensory or mental skills or a lack of experience and/or a lack of knowledge, unless they are supervised by a person who has responsibility for their safety or they receive instructions from this person on how to use the device. Supervise children around the device to ensure they do not play with it.
- Do NOT use the hearing amplifiers in potentially explosive and hazardous environments.
- Do NOT use the devices in oxygen-rich environments and over an extended period of time.
- Please contact the manufacturer in the event of the following:
 - if you require assistance in the set-up, use or maintenance of the hearing amplifiers; or
 - if the devices behave unexpectedly during operation or otherwise.
- If you have any doubts, do NOT use a damaged hearing amplifier; contact your retailer or the specified Customer Services address.
- Switch the devices off immediately if they are faulty or not working properly.

- Do not attempt to open and/or repair the hearing amplifiers yourself. Repairs may only be carried out by Customer Services or authorised retailers. Failure to comply will invalidate the warranty.

Notes on handling batteries:

- If your skin or eyes come into contact with fluid from a battery cell, flush out the affected areas with water and seek medical assistance.
-  Choking hazard! Small children may swallow and choke on batteries. Therefore, store batteries out of the reach of small children.
- If a battery leaks, put on protective gloves and dispose of the device.
- Protect batteries from excessive heat.
-  Risk of explosion! Never throw batteries into a fire.
- Remove the batteries from the device if you do not intend to use the device for an extended period of time.
- Do not disassemble, split or crush the batteries.
- Only use chargers specified in the instructions for use.
- Batteries must be charged correctly prior to use. The instructions from the manufacturer and the specifications in these instructions for use regarding correct charging must be observed at all times.
- Fully charge the battery prior to initial use (see section 6).
- In order to achieve the longest battery service life possible, fully charge the battery at least 4 times a year.

IMPORTANT

- Do NOT wear the hearing amplifiers in the vicinity of strong electromagnetic fields or X-rays.
- Do NOT wear the hearing amplifiers while bathing, showering or swimming.
- Protect the devices from impacts, humidity, dirt, marked temperature fluctuations, direct sunlight and high levels of air humidity.
- High temperatures may develop in closed vehicles during summer. You should therefore never leave your hearing amplifiers inside a vehicle.
- Do NOT drop the devices or subject them to any shocks.
- If the hearing amplifiers fall into water, dry them with a cloth and then have them inspected by your retailer.
- Never attempt to dry the hearing amplifiers in a microwave, a tumble dryer or with a hair dryer. The hearing amplifiers may otherwise become damaged.
- Do NOT use the hearing amplifiers together with a mobile phone. (The radio waves of the mobile phone may cause background noise or a loss of volume.)
- Do NOT wear the hearing amplifiers while applying cosmetics (e.g. make-up, hairspray, perfume or sun cream).
- Do NOT wear the hearing amplifiers while undergoing shortwave therapy or medical treatments that involve strong radio frequency or magnetic fields.

- Take care when operating the buttons. Do NOT press too firmly.
- Do not insert any sharp objects into the microphone opening as this may damage the microphone.
- Do NOT touch the magnets of the hearing amplifier as this may damage the device.
- Do NOT touch the inside of the device. There are many sensitive components that have been precisely configured. Under no circumstances should you poke at the inside of the hearing amplifier with a needle or similar object.
- Service your hearing amplifiers at regular intervals. Make sure there are no foreign objects in between the earplug and speaker, as this could impair the sound.

4. INTENDED USE


The hearing amplifiers are only to be used to help people to hear better. The devices are only intended for the purpose described in these instructions for use. The manufacturer is not liable for damage resulting from improper or careless use.

5. DEVICE DESCRIPTION

The associated drawings are shown on page 3.

- | | |
|-------------------------------|-------------------|
| 1 Earplugs | 7 Connection |
| 2 Earplugs in different sizes | 8 Charger |
| 3 Microphone | 9 Charging slots |
| 4 Volume control | 10 Cleaning brush |
| 5 Power switch (N=On, O=Off) | 11 Storage box |
| 6 Speaker | |

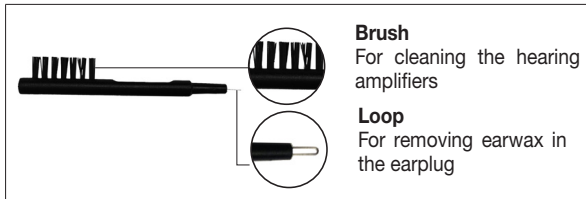Cleaning brush

6. INITIAL USE

Remove the hearing amplifiers from the packaging and check that the devices and all accessories are included and not damaged.

Choosing the right earplugs

Choose an appropriately sized earplug and position it on the speaker.

1. The earplug should sit loosely and comfortably in your ear. Try different earplugs until you find the ones that are most comfortable for you.
2. If the device makes a whistling sound, make sure that the earplug is sitting correctly in your ear. If necessary, use a different earplug. If the volume is too low even at a higher volume setting, the battery may need to be charged. See below for charging instructions.

Using the hearing amplifiers

1. Turn the volume control to the lowest setting.
2. Place the device on your ear and insert the earplug into the ear canal until it fits snugly. Do not force it into your ear.
3. Slide the power switch to the “N” (On) position and turn the volume control to a suitable volume setting.

Charging:

1. Make sure that the device is switched off and then insert it into one of the charger slots so that it clicks into place.

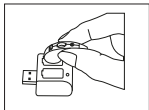
Fig. 1

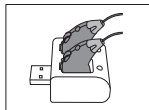
Fig. 2

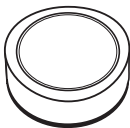
Note: Please ensure that you insert the device correctly into the charger so that it can be charged correctly. The charger is equipped with charging slots that enable you to insert the device in the correct position. Allow the device to click into place in one of these charging slots (Fig. 1) so that the power switch sits in the opening (Fig. 2).

You can charge one or both hearing amplifiers at the same time.

2. Connect the charger to a USB adapter to use it via a socket, or plug the charger directly into a USB port without a USB adapter.
3. The red charging LED lights up during the charging process. When charging is complete, the charging LED lights up green; a full charge usually takes about seven hours.
4. When charging is complete, disconnect the charger from the mains and remove the hearing amplifier(s) from the charger.

Inserting the drying capsule

A drying capsule is included in delivery. Wearing the hearing amplifiers behind your ear means they come into contact with your skin, and therefore also with sweat, for example. As a result, you should dry the devices after each application to care for them properly.

Remove the drying capsule from the packaging and place it in the storage box with the bottom side up.

The capsule has a colour indicator. As the drying capacity decreases over time, the colour changes from bright orange to white. If the capsule is a whitish colour, it no longer has any drying capacity and has to be exchanged.

To find out where to obtain a new drying capsule, see section “Accessories and replacement parts”.

Note: To ensure the best possible drying performance, close the lid of the storage box after placing the hearing amplifiers inside.

7. CLEANING AND MAINTENANCE

IMPORTANT

- Always keep the hearing amplifier clean.
- Always treat the products with care and prevent the earplug and microphone from becoming blocked with foreign matter. This impairs the function of the sound amplification.

- Clean the hearing amplifiers and earplugs with a clean, dry cloth after each use. If they are very dirty, you can also moisten the cloth with a mild soapy solution. Never use abrasive, corrosive or solvent-based cleaning products.
- Under no circumstances should you hold the hearing amplifier under water, as this can cause liquid to enter and damage the hearing amplifier.
- The supplied cleaning brush is suitable for removing earwax stuck in the earplug (see device description).
- Allow the hearing amplifier to dry fully after cleaning. Store the hearing amplifiers in a safe, dry and clean place in the relevant storage box with the drying capsule.
- The drying capsule has to be replaced if the colour of the granulate changes from orange to white (you can find out where to obtain a new drying capsule in section “9. Accessories and replacement parts”).

8. ACCESSORIES AND REPLACEMENT PARTS

Accessories and replacement parts are available from the corresponding listed service address under the stated material number.

Designation	Item number and/or order number
Drying capsules, HA 55/60/80/85 8 pcs.	641.15
Earplugs, HA 55 6 pcs.	164.381

9. WHAT IF THERE ARE PROBLEMS?

Problem	Possible cause	Solution
No tone	Device is not switched on	Turn the power switch to "On"
	Battery is empty	Charge the battery
	Microphone is dirty/ clogged	Clean the microphone
	The device is damaged	Contact Customer Services

Problem	Possible cause	Solution
Low volume	Volume is set too low	Turn the volume control to increase the volume
	Battery is empty	Charge the battery
	Earplug is dirty/ clogged	Clean the earplug
	Microphone is dirty/ clogged	Clean the microphone

Problem	Possible cause	Solution
Volume control is set very high	The hearing amplifier's earplug is not positioned correctly in the ear, meaning the hearing amplifier may not function correctly or an unpleasant whistling sound (acoustic feedback) may be heard.	<ul style="list-style-type: none">- Switch off the hearing amplifier and insert it into your ear. Make sure that the hearing amplifier is inserted correctly.- If the earplug does not fit correctly in your ear, try a different size. If possible, use an earplug that fits correctly. If you have any doubts, please contact Customer Services.

Problem	Possible cause	Solution
The device makes loud whistling noises (acoustic feedback)	The hearing amplifier's earplug is not positioned correctly in the ear, which can cause feedback between the speaker and the microphones.	<ul style="list-style-type: none"> - Position the earplug slightly deeper in the ear or use a larger size. - Check whether there is any hair, part of the outer ear or a glasses arm above the device's microphone opening. This can lead to unpleasant whistling noises.

10. DISPOSAL

General disposal

For environmental reasons, do not dispose of the device in the household waste at the end of its useful life. Dispose of the device at a suitable local collection or recycling point in your country. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.

Battery disposal

- The empty, completely flat batteries must be disposed of using specially designated collection boxes, recycling points or electronics retailers. You are legally required to dispose of the batteries:
- The codes below are printed on batteries containing harmful substances:

Pb = Battery contains lead

Cd = Battery contains cadmium

Hg = Battery contains mercury

11. TECHNICAL SPECIFICATIONS IN ACCORDANCE WITH IEC 60118-7:2005

Type	HA 55
Max. sound pressure level (OSPL90)	$\leq 128 \pm 3$ dB
Sound amplification	$\leq 50 \pm 3$ dB
Frequency range	500–3200 Hz
Overall harmonic distortion	$\leq 10\%$
Power supply	≤ 10 mA
Charging time	7–10 hours
Operation period	16 hours
Dimensions	40 × 13 × 10 mm
Weight	4.3 g

Operating temperature range	0–40°C, relative humidity: ≤ 90%
Transport/storage temperature range	-10–55°C, relative humidity: ≤ 90%

Technical information is subject to change without notification to allow for updates.

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Distributed by: Beurer GmbH • Söflinger Str. 218
Shanghai International Holding Corporation GmbH (Europe)
Huizhou Jinghao Medical Technology Co., Ltd