beurer

FM 90

DE Fußmassagegerät Gebrauchsanweisung	2	IT Apparecchio per massaggio plantare Istruzioni per l'uso.....	45
EN Foot massager Instructions for use	13	TR Ayak masajı cihazı Kullanım kılavuzu	56
FR Appareil de massage des pieds Mode d'emploi.....	23	RU Прибор для массажа ног Инструкция по применению	66
ES Aparato de masaje de pies Instrucciones de uso.....	34	PL Aparat do masażu stóp Instrukcja obsługi.....	76

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Read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

WARNING

- The device is only intended for domestic/private use, not for commercial use.
- This device may be used by children over the age of 8 and by people with reduced physical, sensory or mental skills or a lack of experience or knowledge, provided that they are supervised or have been instructed on how to use the device safely, and are fully aware of the consequent risks of use.
- Children must not play with the device.
- Cleaning and user maintenance must not be performed by children unless supervised.
- If the mains connection cable of this device is damaged, it must be disposed of. If it cannot be removed, the device must be disposed of.
- The device has a hot surface. Persons insensitive to heat must be careful when using the device.
- Do not insert any objects into the device apertures or the rotating parts. Ensure that the moving parts can move freely at all times.
- Under no circumstances should you open or repair the device yourself, as faultless functionality can no longer be guaranteed thereafter. Failure to adhere to this instruction will void the warranty.

Table of contents

1. Included in delivery..... 14	7. Use 18
2. Signs and symbols 14	8. Cleaning and maintenance 20
3. Intended use..... 15	9. What if there are problems? 21
4. Warnings and safety notes 16	10. Disposal..... 21
5. Device description..... 18	11. Technical specifications 21
6. Initial use 18	12. Warranty/service..... 22

1. Included in delivery

Check that the exterior of the delivery cardboard packaging is intact and make sure that all components are present. Before use, ensure that the device and accessories show no visible damage and all packaging material has been removed. If you have any doubts, do not use the device and contact your retailer or the specified Customer Service address.

1 x Foot Massager


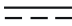
1 x Mains adapter

1 x These instructions for use

2. Signs and symbols

The following symbols appear in the instructions for use and on the type plate

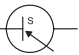	WARNING	Warning notice indicating a risk of injury or damage to health.
	IMPORTANT	Safety note indicating possible damage to the device/accessory.
	Note	Note on important information.
	Only use indoors.	
	Observe the instructions for use	
	Green dot (Der Grüne Punkt): German dual waste collection system	
	Dispose of packaging in an environmentally friendly manner	
	Protection class 2 device	
	Disposal in accordance with the Waste Electrical and Electronic Equipment EC Directive – WEEE	
	Manufacturer	
IP 20	Protected against solid foreign objects 12.5 mm in diameter and larger	
	This product satisfies the requirements of the applicable European and national directives.	
	Energy efficiency level 6	

	Switching mains part
	Short-circuit-proof safety isolating transformer
	Polarity
	The products demonstrably meet the requirements of the Technical Regulations of the EAEU.
	Direct current The device is suitable for use with direct current only
	United Kingdom Conformity Assessed Mark

3. Intended use

This device is solely intended for foot massage on people. The device is only intended for **private use**, and not for medical or commercial purposes. Use the device for the purpose it was developed for only and in the manner specified in these instructions for use. Any form of improper use can be dangerous. The manufacturer is not liable for damage resulting from improper or incorrect use. Do not use the device if one or several of the following notes applies to you. If you are unsure whether the device is suitable for you, please consult your doctor.

- Do not use the foot massager if you suffer from a medical abnormality or injury in the area of your feet (e.g. open wounds, warts, athlete's foot).
- Do not use the foot massager on acutely or chronically diseased, injured or irritated skin (e.g. inflamed skin – whether painful or not, reddened skin, rashes, venous inflammation, arterial thrombosis, venous thrombosis, thrombophlebitis, varicose veins, allergies, burns, bruises, swellings, wounds that are still healing or post-operative scars).
- Only use the foot massager on healthy and clean skin.
- Do not use the foot massager on animals.
- Do not use the foot massager for periods of longer than 15 minutes.
- Do not use the foot massager whilst asleep, driving a vehicle or operating machinery.
- Do not use the foot massager on tissue that is prone to bleeding.
- Never operate the foot massager unsupervised. Supervise children around the device to ensure they do not play with it.
- If you are unsure whether the foot massager is suitable for you, please consult your doctor before use.
- If you suffer from a severe illness or have recently had surgery on your feet, please consult your doctor before using the foot massager.
- If you suffer from diabetes or thrombosis, consult your doctor before using the foot massager.
- If you suffer from sensory impairment that reduces the feeling of pain (e.g. metabolic disorders), please consult your doctor before use.
- If you are suffering from pain, the cause of which is unknown, consult your doctor before using the foot massager.

4. Warnings and safety notes

Read these instructions for use carefully! Non-observance of the following information may result in personal injury or material damage. Store these instructions for use and make them accessible to other users. Make sure you include these instructions for use when handing over the device to third parties.

WARNING

Keep the device and packaging material away from children. The mains adapter cable poses a risk of strangulation and there is a risk of choking due to small parts that can be swallowed.

Electric shock

WARNING

As with every electrical device, use this device carefully and cautiously, in order to avoid hazards due to electric shock.

The device should therefore:

- Only be operated with the voltage specified on the device
- Never be used if it or its accessories show visible signs of damage
- Not be used during a storm.

In the event of defects or operational faults, switch the device off immediately and disconnect it from the mains socket. Do not pull on the mains cable or on the device to disconnect the plug from the socket. Never hold or carry the device by the mains cable. Keep the cables away from hot surfaces. Never open the device. Ensure that the device and the mains cable do not come into contact with water, steam or other liquids.

Therefore, only use the device:

- In dry interior rooms (e.g. never in the bath or sauna)
- With dry hands.

Do not pull, twist or bend the mains cable. Do not insert needles or any sharp objects. Ensure that the mains cable is not routed or pulled over pointed or sharp objects.

Never reach for a device that is submerged in water. Disconnect it from the mains plug immediately. Do not use the device if it or its accessories are visibly damaged. Do not drop the device or subject it to any impacts.

Repairs

WARNING

Only specialist personnel may perform repairs on electrical devices. Improper repairs may subject users to considerable danger. For repairs, please contact Customer Services or an authorised retailer.

Risk of fire

WARNING

There is a risk of fire if the device is used improperly or these instructions for use are ignored. Therefore:

- Never operate the foot massager unsupervised, especially if there are children nearby
- Never operate the foot massager underneath a cover, such as a blanket, pillow, etc.
- Never operate the foot massager near petrol or other highly flammable substances

Handling

IMPORTANT

The device must be switched off and disconnected after every use and before cleaning.

- Do not place your full weight (sitting, lying down, or standing) on the moving parts of the device or place any objects on the device.
- Do not expose the device to direct sunlight or high temperatures.
- Protect the device from dust, dirt and humidity.
- Never submerge the device and the mains adapter in water or other liquids.


Damage

WARNING

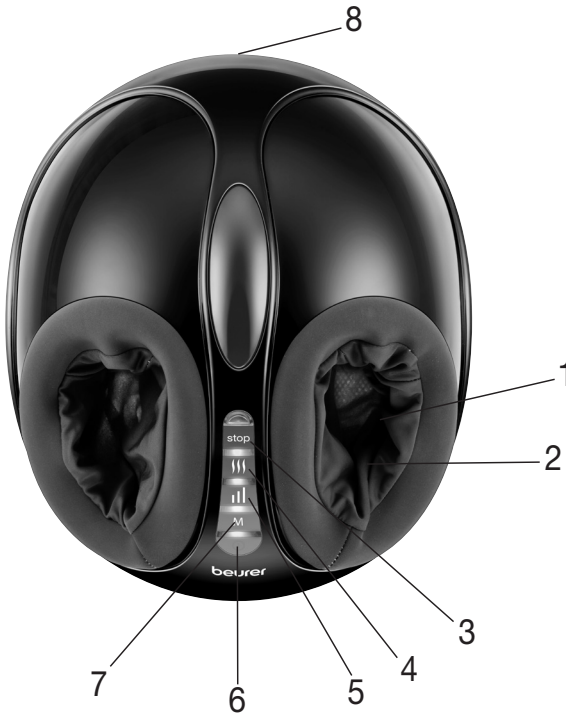- Only specialist personnel may perform repairs on electrical devices. Improper repairs may subject users to considerable danger. For repairs, please contact Customer Services or an authorised retailer.
- If the device is damaged, do not use it and contact your retailer or the specified Customer Services address.
- To ensure that the device functions effectively, do not drop it or dismantle it.
- Check the device for signs of wear and tear or damage. If there are such signs of wear and tear or damage or if the device was used improperly, it must be returned to the manufacturer or retailer before further use.
- Switch the device off immediately if it is faulty or not working properly.
- Never attempt to open and/or repair the device yourself. Repairs may only be carried out by Customer Services or authorised retailers. Failure to comply with this instruction will void the warranty.
- The manufacturer is not liable for damage resulting from improper or incorrect use.

5. Device description

1. Massage surface with massage heads
2. Removable cover (washable)
3. Stop button 
4. Heat button 
5. Intensity button 
6. ON/OFF button 
7. Mode button 
8. Connection socket for mains adapter


6. Initial use

Connecting the mains adapter

To operate the foot massager with the supplied mains adapter, you need to connect the mains adapter to the foot massager first. Proceed as follows:

1. Remove all packaging material.
2. Check the foot massager, the mains adapter and the mains cable for damage.
3. Connect the mains adapter cable to the connection socket on the underneath of the foot massager.
4. Insert the mains adapter into a suitable socket.

 **WARNING** Trip hazard! Ensure that the mains cable does not pose a trip hazard.

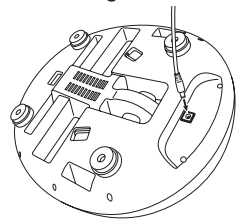7. Use





 **WARNING**




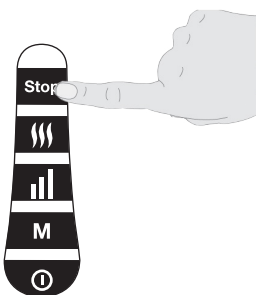
- Only use the foot massager when you are sitting down. Never use the foot massager when standing up.
- Use the massager for max. 15 minutes. Longer massages can lead to over-stimulation of the muscles, creating tension instead of aiding relaxation.

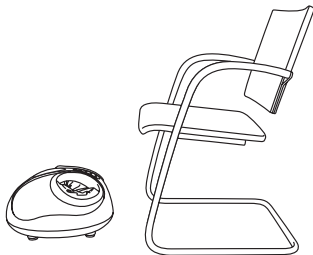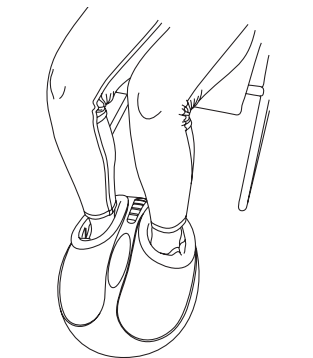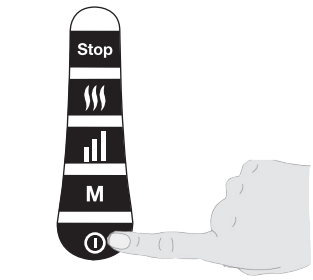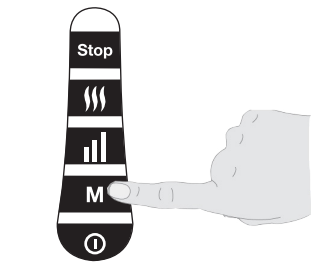i Note

The device features an automatic switch-off function, which is set to the maximum operating duration of 15 minutes.

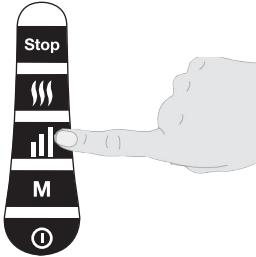To start foot stimulation, proceed as follows:
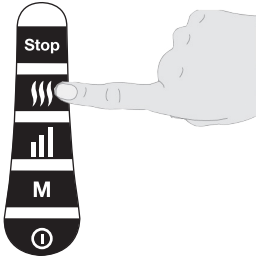
<p>Step 1: Place the foot massager on the floor</p> <ul style="list-style-type: none">Place the foot massager on an even, hard surface. Position a chair in front of the foot massager so that you can reach the device comfortably with your feet when sitting in the chair.	
<p>Step 2: Place your feet in the foot massager</p> <p>i Note</p> <p>You can use the foot massager either barefoot or with socks.</p> <ul style="list-style-type: none">Slip your feet into the foot massager. Ensure that the soles of your feet are placed on the massage heads.	
<p>Step 3: Switch on the foot massager i</p> <p>Press the ON/OFF button i to switch the foot massager on. The LED on the ON/OFF button begins to light up green. The foot massager starts in mode 1 and the lowest intensity.</p>	
<p>Step 4: Set the mode M</p> <p>The foot massager has three different massage modes. Select your desired mode using the mode button M.</p> <p>Mode 1: LED green = Only air pressure Mode 2: LED blue = Massage without air pressure Mode 3: LED red = Massage with air pressure</p>	
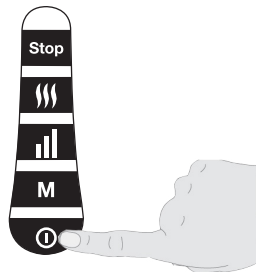
<p>Step 5: Set the intensity </p> <p>For the air pressure massage, the foot massager has three intensity levels (low, medium, high). Select your desired intensity level using the intensity button .</p> <p>LED green = low intensity LED blue = medium intensity LED red = high intensity</p>	
<p>Step 6: Switch the heat on/off </p> <p>The foot massager has an optional heat function. You can warm your feet during the massage by using the heat function.</p> <ul style="list-style-type: none"> To switch on the heat function, press the heat button  during the massage. The LED on the heat button begins to light up red. Press the heat button again to switch off the heat function. The LED on the heat button goes out. 	
<p>Step 7: Stop the foot massage</p> <ul style="list-style-type: none"> If, during use, you experience an excessive or unpleasant feeling of pressure, press the start/stop button to stop the treatment. To start the foot massager again, press the ON/OFF button . 	
<p>Step 8: Switch off the foot massager</p> <ul style="list-style-type: none"> To switch off the foot massager, press the ON/OFF button  when the device is switched on. The LED on the ON/OFF button  goes out. 	

8. Cleaning and maintenance

Cleaning the foot massager

WARNING

- Unplug the foot massager each time before cleaning.
- Clean the foot massager using only the methods specified. Under no circumstances may liquid enter the foot massager or the accessories.
- Clean the foot massager with a slightly damp cloth. If it is very dirty, you can also moisten the cloth with a mild soapy solution.

Cleaning the covers

The covers of the massage rollers can be washed by hand. Proceed as follows:

1. To remove the covers from the foot massager, undo the hook-and-loop fastener of the covers.
2. Wash the covers by hand.
3. Allow the covers to dry. Only insert the covers into the foot massager again when they are completely dry.
4. Ensure that the hook-and-loop fasteners of the covers are firmly attached to the foot massager.

Storage

If you do not plan to use the foot massager for a long period of time, we recommend that you store it in the original packing in a dry environment, ensuring it is not weighed down by additional items. Store the foot massager away from children and pets. Do not place any objects on the foot massager.

9. What if there are problems?

Problem	Cause	Solution
The massage heads are rotating too slowly	The load on the massage heads is too great.	Free up massage heads.
The massage heads do not move	The device is not connected to mains power supply.	Connect the mains plug and switch on the device.
	Overheating protection triggered.	Disconnect the device from the power supply, wait at least 15 minutes, then plug in again and switch on.

10. Disposal

For environmental reasons, do not dispose of the device in the household waste at the end of its service life. Dispose of the device at a suitable local collection or recycling point in your country. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.

11. Technical specifications

Mains operation: Input Output	100-240 V~, 50/60 Hz, 1.5 A 24.0 V === 2.0 A, 48.0 W
Average active efficiency:	≥ 89.35 %
Efficiency at low load (10 %):	≥ 87.87 %
No-load power consumption:	≤ 0.08 W
Dimensions:	36 x 42 x 25 cm
Weight:	approx. 4.5 kg

12. Warranty/service

Beurer GmbH, Söflinger Straße 218, 89077 Ulm, Germany (hereinafter referred to as “Beurer”) provides a warranty for this product, subject to the requirements below and to the extent described as follows.

The warranty conditions below shall not affect the seller’s statutory warranty obligations which ensue from the sales agreement with the buyer.

The warranty shall apply without prejudice to any mandatory statutory provisions on liability.

Beurer guarantees the perfect functionality and completeness of this product.

The worldwide warranty period is 3 years, commencing from the purchase of the new, unused product from the seller.

The warranty only applies to products purchased by the buyer as a consumer and used exclusively for personal purposes in the context of domestic use.
German law shall apply.

During the warranty period, should this product prove to be incomplete or defective in functionality in accordance with the following provisions, Beurer shall carry out a repair or a replacement delivery free of charge, in accordance with these warranty conditions.

If the buyer wishes to make a warranty claim, they should approach their local retailer in the first instance: see the attached “International Service” list of service addresses.

The buyer will then receive further information about the processing of the warranty claim, e.g. where they can send the product and what documentation is required.

A warranty claim shall only be considered if the buyer can provide Beurer, or an authorised Beurer partner, with

- a copy of the invoice/purchase receipt, and
- the original product.

The following are explicitly excluded from this warranty:

- deterioration due to normal use or consumption of the product;
- accessories supplied with this product which are worn out or used up through proper use (e.g. batteries, rechargeable batteries, cuffs, seals, electrodes, light sources, attachments and nebuliser accessories);
- products that are used, cleaned, stored or maintained improperly and/or contrary to the provisions of the instructions for use, as well as products that have been opened, repaired or modified by the buyer or by a service centre not authorised by Beurer;
- damage that arises during transport between manufacturer and customer, or between service centre and customer;
- products purchased as seconds or as used goods;
- consequential damage arising from a fault in this product (however, in this case, claims may exist arising from product liability or other compulsory statutory liability provisions).

Repairs or an exchange in full do not extend the warranty period under any circumstances.

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