beurer

MG 35 Deep Roll

DE	Massagerolle mit Vibration Gebrauchsanweisung.....	2
EN	Vibrating massage roller Instructions for use.....	29
FR	Rouleau de massage à vibration Mode d'emploi	55
ES	Rodillo de masaje con vibración Instrucciones de uso	81
IT	Rullo massaggiante con vibrazione Istruzioni per l'uso	107
TR	Titreşimli masaj silindiri Kullanım kılavuzu	133
RU	Роликовый вибромассажер Инструкция по применению ...	158
PL	Rollka do masażu z wibracjami Instrukcja obsługi	181

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ENGLISH

Read these instructions for use carefully and keep them for later use; be sure to make them accessible to other users and observe the information they contain.

WARNING

- The device is intended only for domestic/private use, not for commercial use.

- This device can be used by children over the age of 8 and by people with reduced physical, sensory or mental skills or a lack of experience or knowledge, provided that they are supervised or have been instructed on how to use the device safely and are aware of the consequent risks of use.
- Children must not play with the device.
- Cleaning and user maintenance must not be performed by children unless supervised.
- Under no circumstances should you open or repair the device yourself, as faultless functionality can no longer be guaranteed thereafter. Failure to comply will invalidate the warranty.

Dear customer,

Thank you for choosing a product from our range. Our name is synonymous with high-quality, thoroughly tested products for applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage, beauty, baby and air.

With kind regards,

Your Beurer team

Table of contents

1. Getting to know your device	32
2. Included in delivery	32
3. Signs and symbols	33
4. Proper use.....	35
5. Warnings and safety notes	36
6. Device description.....	41
7. Initial use	42
8. Usage.....	43
9. Cleaning and maintenance	48
10. What if there are problems?	49
11. Disposal.....	50
12. Technical specifications	51
13. Warranty/service	51

1. Getting to know your device

The MG 35 vibrating massage roller allows you to carry out a targeted trigger point massage of tense muscle groups. It can also help to loosen fascial tissue and break down fasciae agglutinations. The massage roll is comfortable to use thanks to its soft-touch surface. The massage roll has three moving parts so that it can adapt flexibly to your body movements.

The massage roll has three different intensity levels and a pre-set massage mode for tailored massage according to your personal preferences.

2. Included in delivery

1 x vibrating massage roller

1 x USB charging cable

1 x set of these instructions for use

3. Signs and symbols

The following symbols are used on the device, in these instructions for use, on the packaging and on the type plate for the device:

	WARNING	Warning notice indicating a risk of injury or damage to health.
	IMPORTANT	Safety note indicating possible damage to the device/accessory.
	Note	Note on important information.
	Read the instructions for use	
	CE labelling	This product satisfies the requirements of the applicable European and national directives.
	Disposal in accordance with the Waste Electrical and Electronic Equipment EC Directive – WEEE	

 <p>Pb Cd Hg</p>	<p>Do not dispose of batteries containing harmful substances with household waste</p>
	<p>Manufacturer</p>
	<p>Dispose of packaging in an environmentally friendly manner</p>
	<p>The products demonstrably meet the requirements of the Technical Regulations of the EAEU.</p>
	<p>Direct current</p>
 <p>PIN1 “+” PIN4 “-”</p>	<p>Polarity</p>

4. Proper use

WARNING

The device is suitable only for massage treatment of the thighs, calves, shoulders, arms and back region (excluding the spine). The device is not a medical device; rather it is a massage treatment device.

The device is only intended for private use, and not for medical or commercial purposes. The device may only be used for the purpose for which it is designed and in the manner specified in the instructions for use. Any improper use may be dangerous. The manufacturer is not liable for damage resulting from improper or incorrect use. Do not use the massager if one or several of the following warnings applies to you. If you are unsure whether the device is suitable for you, please consult your doctor.

5. Warnings and safety notes

WARNING

- Do NOT use the device on animals.
- Do NOT use the device if you suffer from a medical abnormality or injury in the area to be massaged (e.g. slipped disc, open wounds).
- Do NOT use the device on the area around the heart.
- Do NOT use the device on swollen, burnt, inflamed or injured skin and body parts.
- Do NOT use the device on cuts, capillaries, varicose veins, acne, couperose, herpes or other skin diseases.
- Do NOT use the device during pregnancy.
- Do NOT use the device on the head.
- Do NOT use the device on the face (eyes), the larynx or other particularly sensitive body parts.
- Do NOT use the device in the pubic area.
- Do NOT use the device while sleeping.
- Do NOT use the device after the consumption of medicines or alcohol (reduced cognition!).
- Do NOT use the device in a vehicle.

You should consult your doctor before using the device, especially

- If you suffer from a severe illness
- If you have a pacemaker, implant or other aids
- If you suffer from thrombosis
- If you suffer from diabetes
- if you are from suffering pain, the cause of which is unknown

WARNING

- Keep packaging material away from children. There is the risk of suffocation.

There is a risk of fire if the device is used improperly or these instructions for use are ignored.

Therefore, the device must

- Never be left lying around unattended, especially if there are children nearby
- Never be used underneath a cover, such as a blanket, pillow, etc.
- Never be used near petrol or other highly flammable substances

- Only be used in dry interior rooms (e.g. never in the bath or sauna)

Make sure that the device does not come into contact with water or other liquids.

Never reach for a device that is submerged in water.

IMPORTANT

- Before use, ensure that the device and accessories show no visible damage. If you have any doubts, do not use the device and contact your retailer or the specified Customer Service address.
- Never open the housing.
- Keep the device away from pointed or sharp objects.
- If the device has been dropped or has suffered any other damage, it must no longer be used.

Notes on handling batteries

WARNING

- If your skin or eyes come into contact with fluid from a battery cell, flush out the affected areas with water and seek medical assistance.
- Protect batteries from excessive heat.
-  **Risk of explosion!** Never throw batteries into a fire.
- Do not disassemble, split or crush the batteries.
- Only use chargers specified in the instructions for use.
- Batteries must be charged correctly prior to use. The instructions from the manufacturer and the specifications in these instructions for use regarding correct charging must be observed at all times.
- Fully charge the battery prior to initial use (see section on “Initial use”).
- In order to achieve as long a battery service life as possible, fully charge the battery at least 2 times a year.
- Only charge the device using the supplied charging cable.
- Always switch the device off in advance of charging.

Repairs

WARNING

- Only specialist personnel may perform repairs on electrical devices. Improper repairs may subject users to considerable danger. For repairs, please contact Customer Services or an authorised retailer.

6. Device description

1 ON/OFF button

2 LED display (3 LEDs)

Left LED lights up = Low intensity level

Centre LED lights up = Medium intensity level

Right LED lights up = High intensity level

All LEDs light up = Pre-set massage mode


3 Connection for USB charging cable

7. Initial use


Note

Before using the massage roll for the first time, charge it first for a minimum of 2 ½ hours. Proceed as follows:

1. Connect the supplied USB charging cable to the massage roll. The connection for the USB charging cable is located on the side of the massage roll.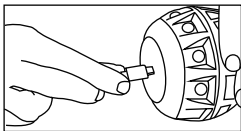

2. Then plug the USB charging cable into a PC or, using a USB adapter, into a suitable socket.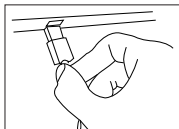

Whilst charging, the LEDs flash from left to right. As soon as the massage roll is fully charged, all three LEDs are permanently illuminated. When fully charged, the battery will last for approx. two hours of operation at a low intensity level or for one hour of operation at a high intensity level. You cannot use the massage roll during the charging process. If the battery is dead, the LEDs flash in the relevant set intensity level.

8. Usage

8.1 Notes on use

Note

The massage should feel pleasant and relaxing at all times. If the massage is painful or uncomfortable, stop the massage or change your position or the contact pressure.

A slight haematoma may well occur – this is a positive therapeutic effect as the use of the massage roll can selectively encourage your blood flow. In isolated cases, areas of redness may also occur on the skin. If you experience excessive skin irritation, stop using the device and consult a doctor.

We recommend treating each area (lower body, mid-body and upper body) three times a week for 15 minutes each time. When carrying out a session, we recommend training all three areas. However, you may also choose to focus on one of the above areas. Carry out treatment consciously and slowly.

8.2 Switching on/selecting the intensity level

Note


The massage roll has an automatic switch-off function. This means that the massage roll automatically switches off after a period of use of 15 minutes.

1. Briefly press the **ON/OFF button** to switch on the massage roll. The LED display starts to light up and the massage roll starts to vibrate at the lowest intensity level.
2. The massage roll has a total of three intensity levels (low, medium and high) and a massage mode (intermittent). To switch between the four settings, briefly press the **ON/OFF button**. Select one of the four settings depending on your personal preferences. The massage roll switches off again if you press the **ON/OFF button** again in the fourth setting. You can also switch off the massage roll directly during usage by pressing and holding the **ON/OFF button** for 2 seconds. The LED display goes out and the vibration stops.

8.3 Examples of use:


Calf

Sit on the floor and place one leg on the massage roll so that it is positioned on the bottom half of the calf. Support yourself with your hands while leaning back and lift your hips. Tense your stomach and bottom muscles and shift your lower body forwards and backwards. The calf muscles are stimulated.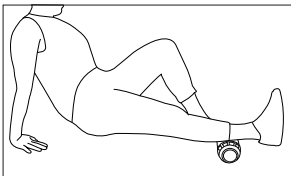

Thigh/bottom


Lie on your side on the floor and stretch both legs straight out. Bend one arm and support yourself on the floor with your elbow while keeping your other arm relaxed by your side. Place the massage roll under your hips. Now roll over the massage roll with slow movements. Vary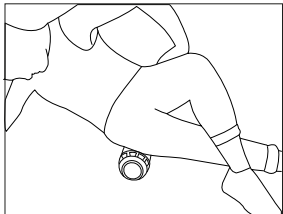

the exercise by supporting yourself with both arms while leaning back; then roll your bottom over the massage roll with gentle pressing movements.


Thigh

Sit upright on the floor or on a chair and hold the massage roll by the two outer parts and now roll it over your thigh slowly and using some pressure. The centre, flexible element of the roll should now massage your thigh muscle.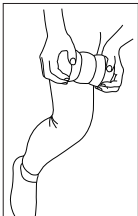

Arm/shoulder

Lean against a wall while standing up, with your side facing the wall. Place the massage roll between your upper arm and the wall. Now apply gentle pressure on the massage roll and move up and down with your legs so that the massage roll rolls slowly along your arm. Increase the distance between your legs and the wall in order to increase the pressure on the massage roll. Now slowly turn your back to the wall so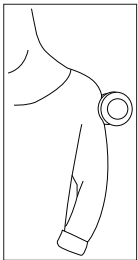

that the massage roll moves along your shoulder area and the muscles in your shoulders are stimulated.


Back

WARNING

Do not move the massage roll directly over bone.

Lie on your back. Place the massage roll to the right or left of the spine between your back and the floor. Bend your legs. Now move slowly up and down and roll the massage roll over your back.

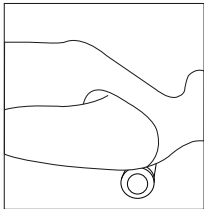9. Cleaning and maintenance

Cleaning

WARNING

- Cleaning must NOT be performed by children unless supervised.
- Switch off the device each time before cleaning.
- Clean the device with a slightly damp cloth only.
- Clean the device using only the methods specified. Under no circumstances may liquid enter the device.
- Do not use the device again until it is completely dry.
- Do not clean the device in a dishwasher.
- Do not use any abrasive cleaning products or hard brushes.

Care

If you do not plan to use the device for a long period of time, we recommend that you store it in the original packaging in a dry environment, ensuring it is not weighed down by additional items.

10. What if there are problems?

Problem	Solution
The massage roll does not roll evenly during usage.	Try to avoid applying load on one side and, as far as possible, you should apply load to the massage roll evenly.
LEDs do not flash during charging.	Check whether the USB charging cable is correctly connected to the massage roll.

11. Disposal

IMPORTANT

- Batteries must not be disposed of in the household waste. They may contain poisonous heavy metals and are subject to special refuse treatment.
- The codes below are printed on rechargeable batteries containing harmful substances:

Pb = Battery contains lead

Cd = Battery contains cadmium

Hg = Battery contains mercury

For environmental reasons, do not dispose of the device in household waste at the end of its service life. Dispose of the device at a suitable local collection or recycling point in your country. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment).

If you have any questions, please contact the local authorities responsible for waste disposal.

12. Technical specifications

Dimensions	15.6 x 8.7 x 8.7 cm
Weight	Approx. 746 g
<u>Battery:</u>	
Capacity	1800 mAh
Nominal voltage	3.7 V
Type designation	Lithium-ion

13. Warranty/service

Beurer GmbH, Söflinger Straße 218, 89077 Ulm, Germany (hereinafter referred to as “Beurer”) provides a warranty for this product, subject to the requirements below and to the extent described as follows.

The warranty conditions below shall not affect the seller’s statutory warranty obligations which ensue from the sales agreement with the buyer.

The warranty shall apply without prejudice to any mandatory statutory provisions on liability.

Beurer guarantees the perfect functionality and completeness of this product.

The worldwide warranty period is 3 years, commencing from the purchase of the new, unused product from the seller.

The warranty only applies to products purchased by the buyer as a consumer and used exclusively for personal purposes in the context of domestic use.

German law shall apply.

During the warranty period, should this product prove to be incomplete or defective in functionality in accordance with the following provisions, Beurer shall carry out a repair or a replacement delivery free of charge, in accordance with these warranty conditions.

If the buyer wishes to make a warranty claim, they should approach their local retailer in the first instance: see the attached „International Service“ list of service addresses.

The buyer will then receive further information about the processing of the warranty claim, e.g. where they can send the product and what documentation is required.

A warranty claim shall only be considered if the buyer can provide Beurer, or an authorised Beurer partner, with

- a copy of the invoice/purchase receipt, and
- the original product.

The following are explicitly excluded from this warranty:

- deterioration due to normal use or consumption of the product;
- accessories supplied with this product which are worn out or used up through proper use (e.g. batteries, rechargeable batteries, cuffs, seals, electrodes, light sources, attachments and nebuliser accessories);
- products that are used, cleaned, stored or maintained improperly and/or contrary to the provisions of the instructions for use, as well as products that have been opened, repaired or modified by the buyer or by a service centre not authorised by Beurer;
- damage that arises during transport between manufacturer and customer, or between service centre and customer;
- products purchased as seconds or as used goods;

- consequential damage arising from a fault in this product (however, in this case, claims may exist arising from product liability or other compulsory statutory liability provisions).

Repairs or an exchange in full do not extend the warranty period under any circumstances.

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Beurer GmbH • Söflinger Straße 218 • 89077 Ulm, Germany